INSTRUCTIONS FOR USE

Detachable endo pocket
SB534,SB936, SB957, SB979,
SB1036, SB1057, SB1079

GB Detachable endo pocket

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Read IFU
before use

Single use
only

Do not use device
if packaging is
damaged

Rx only

2460

Distributed by:
CONMED Corporation
525 French Road, Utica,NY 13502
(315) 797-8375 • FAX 1-800-438-3051
Customer Service (USA) 1-800-448-6506
e-mail: info@conmed.com
For International orders or inquiries,
please contact CONMED International Sales
+1 (315) 797-8375 • FAX +1 (315) 735-6235
www.conmed.com

Unimax Medical Systems Inc.
8F-2, No. 127, Lane 235, Pao Chiao Rd, Hsin Tien, Taipei, Taiwan

Issued date: 2017/09/01; Rev. 7

GB ENGLISH

◆Description:

The Unimax Specimen Retrieval System - Detachable Endo Pocket is supplied with a cylindrical tube and cone pouch for ease of insertion through a standard 10,11,12, or 15 mm trocar sheath.

◆Indications:

The Unimax Specimen Retrieval System - Detachable Endo Pocket is indicated for use as a receptacle for the collection and extraction of tissue, organs and calculi during general and laparoscopic surgical procedures.

◆Contraindications:

This device is not intended for use when endoscopic techniques are contraindicated.

◆Adverse Reactions:

Keep the pocket out of sharp instruments or edges. Pocket rupture may result in spillage of fluid or tissue specimen.

◆Preparation:

- ① Insert the Detachable Endo Pocket through a trocar sleeve.
- ② Hold the Detachable Endo Pocket in a syringe-like fashion and push the mobile handle (1) completely forward. The self-opening designed pocket will be deployed and advanced into the abdominal cavity. When the pocket is deployed, it will begin to unroll; the pocket may be further unrolled by using the tip of a blunt laparoscopic instrument.
- ③ Place desired specimen into the pocket. Ensure the entire specimen fits within the confines of the pocket.
- ④ Once all the desired specimen is within the pocket, pull the mobile handle (1) back until the pocket is completely closed.
- ⑤ Draw the mobile handle (1) out from the cylindrical tube so that it is entirely detached. Then there will be 1~2 inches of string exposed on the end of fixed handle (2).
- ⑥ Remove the fix handle (2) from the trocar site. Then the string will be left in the trocar cannula.
- ⑦ Remove the endo pocket at the end of the laparoscopic procedure. When removal, pull the pocket string out of the trocar until the leading edge of the pocket is inside the tip of the trocar. Grasp the string and the trocar and remove from the access site together, the Detachable Endo Pocket with specimen should be removed through the access site easily. If the pocket with specimen cannot be removed, enlargement of the access site is recommended for easier removal.
- ⑧ Upon completion of the endoscopic procedure, dispose the device in accordance with local regulations..

◆Cautions/ Warnings:

- This device is EO sterilized and for single use only. Please do not re-use or re-process to avoid irreversible injury such as cross-infection or injury caused by device malfunction.
- Do not use when the package is opened or damaged.
- Store this device in a dry, clean and safe place. Keep the device away from flammable and combustible chemicals or materials.
- Please follow universal guidelines upon medical waste disposal to avoid any potential cross-infection, environmental contamination or injury caused by sharps waste.

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